




Safepad™ - User instructions

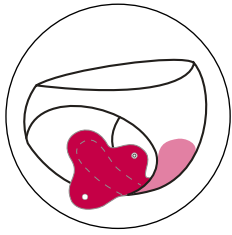


Place Safepad™ inside your underwear

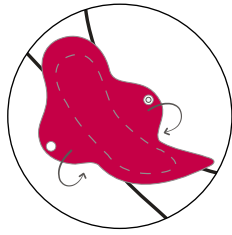

Bend the wings of Safepad™ downward

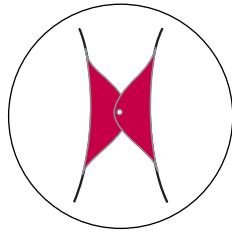


Snap the button in the wings

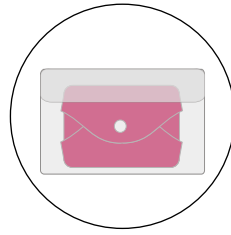


After use fold the Safepad™ and put it into the storage bag until washing


Safepad™ - Washing instructions

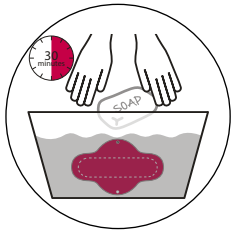


Soak the used Safepad™ in cold water for 30min with detergent or soap. Do not use bleach.

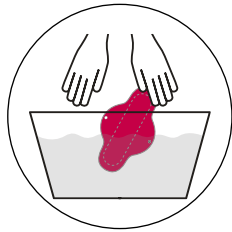


Rinse with clean water

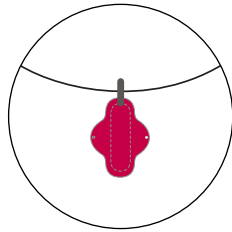

Dry in a well ventilated area

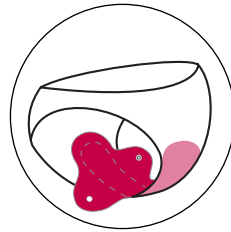

Reuse Safepad™ after it has been dried well.

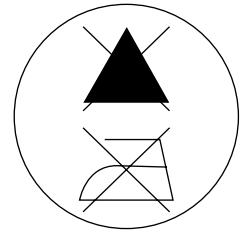

Safepad™ should not be bleached or ironed

Safepad™ - Packaging Content

Standard:

- 3 x Super daypads
- 1 x Long nightpad
- 1 x Storage bag